



*"We are presenting cast products!"*

# Thermoplast Catalogue



1st Floor, Murlidhar Complex,  
Opp. Railway Station, Near  
Hotel Sheelal, Surat, Gujarat-India



+91 7016902738  
0261 2424664




[www.nectarmeditech.com](http://www.nectarmeditech.com)  
[info@nectarmeditech.com](mailto:info@nectarmeditech.com)

First-time specific cast product launch in the Indian market.  
All products are designed to meet the needs of orthopedic surgeons.  
Featuring new innovations and concepts in casting.


## BELOW ELBOW (BE) SLAB




**VOLAR SLAB**

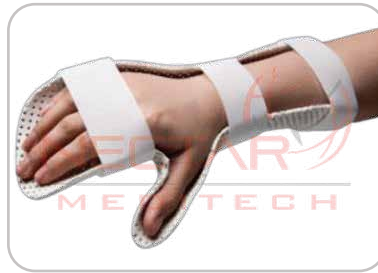

**COCK UP SLAB**

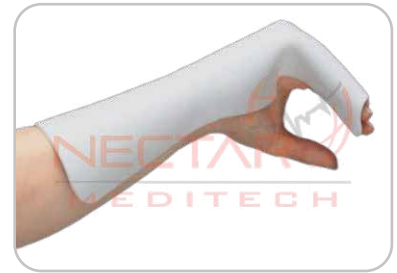

**ULNAR GUTTER**

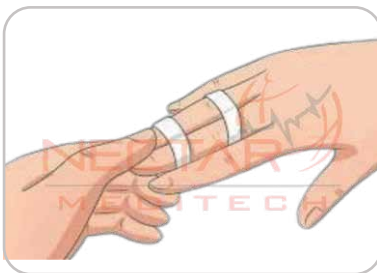

**BUDDY STRAPPING**



**FINGER STRAPPING**



**THUMB SPICA SPLINT**

## BENEFITS OF BE SLAB

- Easy to apply.
- Economic compared to conventional cost.
- Easily clean with tapwater.
- Reusable.
- Good working time interval once soft.
- Can be easily cut it with simple scissor.
- Not create any type of crease in curve that usually seen in conventional slab.
- All types of below elbow application can be made by be slab.



(BELOW KNEE/ABOVE ELBOW)

## BENEFITS OF BK/AE SLAB

- Easy to apply.
- Economic compared to conventional cost.
- Easily clean with tapwater.
- Reusable.
- Good working time interval once soft.
- Can be easily cut it with simple scissor.
- Not create any type of crease in curve that usually seen in conventional slab.
- All types of below elbow application can be made by be slab.

## BENEFITS OF FOREARM SLAB



- Radiolucent properties thta does interfere with radiological imaging.
- Light in weight, nearly 190GM for longest size.
- Lesser changes of atrophy that usually seen in conventional cast.
- Less time required to apply the cast.
- Quality of life for patient will be more compared to conventional cast.
- Rashed and itches free.
- Can be easily remove without use any other external cast cutter or spreader.
- 100% waterproof.




## BENEFITS OF FINGER SPLINT

- | This finger splint is a new concept splint that is anatomic in design and perfectly fits in the finger.
- | Finger splint is very small in size and lightweight as conventional aluminum padded splints are very bulky and big in size.
- | It's 100% waterproof and easy to clean.
- | Easy to wear and remove.
- | Main benefits are contoured with the finger so it does not interfere with side fingers that are usually seen in conventional splints.

### | SIZE CHART

SMALL | MEDIUM | LARGE | EX-LARGE

## BENEFITS OF NASAL SPLINT

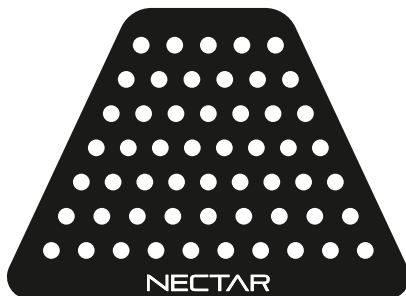

- | Nasal splint is new concept splint that is anatomically designed and perfectly fits in nose size.
- | Nasal splint is very thin and lightweight as conventional aluminum padded splints are very bulky and big in size.
- | Its 100% waterproof and easy to clean.
- | Easy to wear and remove.

\* For more information, please refer to our nasal splint product brochure

## HOW TO USE

- I First cover the injured body part with soft roll, padding, and stockinette.
- I Then soften the front portion of slab with heat and applied slab as per need.
- I Slab will be soft for 1.5 min.
- I If you want to cut extra you can easily cut out with normal scissor only when slab is soft.
- I After getting complet desire shape and size, just place on arm or leg, and then wrap with normal cotton bandage, crepe bandage or surgical tape.

## ATTENTION

- I Don't heat more than 80° C.
- I Don't let stay in hot water more than 5 minutes.
- I Storage in room temperature.

## MATERIAL DETAIL

- I Thermoplastic material.
- I Biodegradable.
- I Soften at 60° c.

### Phone:

+91 7016902738  
0261 2424664

### Mail:

info@nectarmeditech.com  
export@nectarmeditech.com

### Address:

1st Floor, Murlidhar Complex, Opp. Railway  
Station, Near Hotel Sheelal, Surat, Gujarat-India

### Website

www.nectarmeditech.com

Thanks for Watching!