



**M E D I T E C H**

Quality | Service | Innovation



# Nasal Splint



1st Floor, Murlidhar Complex,  
Opp. Railway Station, Near  
Hotel Sheelal, Surat, Gujarat-India




+91 7016902738  
0261 2424664



[www.nectarmeditech.com](http://www.nectarmeditech.com)  
[info@nectarmeditech.com](mailto:info@nectarmeditech.com)

## INDICATION

- I Nasal Bone Fracture
- I Post-Septoplasty
- I Post-Rhinoplasty

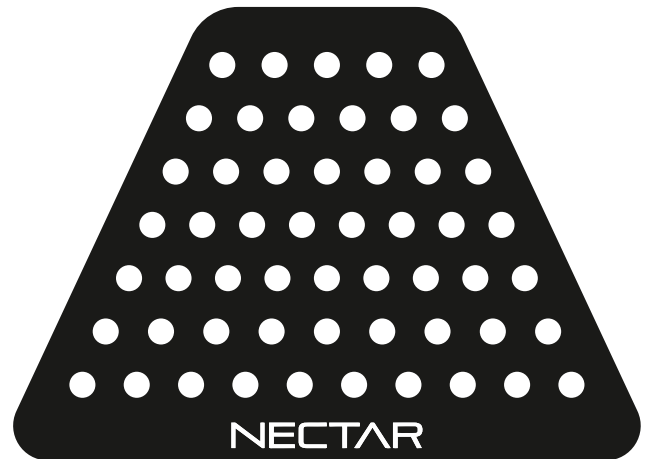

## PRODUCT DESCRIPTION

- I This nasal splint new concept splint that is anatomic in design and perfectly fit in nose size.
- I External nose splints consist of a bandage that cover nose.
- I This nasal splint is vary thin and light weight as conventional aluminium padded splint are very bulky and big in size.
- I It is 100% waterproof and easy to apply and remove.
- I It's usually held in place with surgical tape.

## PRODUCT MATERIAL DETAIL

- I Thermoplastic material
- I Biodegradable



## HOW TO USE

- Put the nasal splint into 60-65 °C hot water for 40 Sec.-01 minute.
- After it is activated, splint will could become soft and pliable, put it back and dry it with a soft cloth.
- After that put splint on the nose. Push it tenderly as per required shape.
- Within 1 minute, Splint will become rigid and fixed.

## ATTENTION

- Don't put the nasal splint into hot water more than 80 °C , storage in room temperature.
- Don't let the nasal splint stay in hot water more than 5 minutes.

## SIZE

S: 45" X 70" X 2.1" ■

M: 50" X 75" X 2.1" ■

L: 55" X 80" X 2.1" ■

(All sizes are in mm format)

## CODE

■ NS-4570

■ NS-5075


■ NS-5580

(Length x Width x Height)

# TWO COLORS

I White

I Skin Color

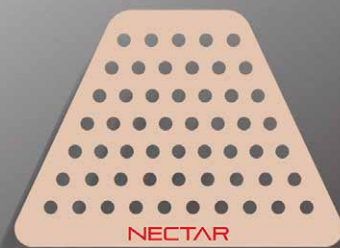

*Thank You!*